



---

# Quick Draw Collection Method



Instructions For Use ►



## At-Home Lab Testing, Let's Do This!



### Important Reminders Before Testing



**Prepare to return your kit the same day you collect your sample.**



**Do not test on a Friday, Saturday, Sunday, or Holiday.**



You will be notified via email when your test results are ready. You can access your results by logging into your Patient Dashboard at: [kits.mylabsdirect.com](https://kits.mylabsdirect.com)



## STEP 1

# Ready to test!

Fill out the Test Kit ID sticker on biohazard bag with black ballpoint pen. Be sure handwriting is legible. Register your kit by scanning the QR code below or

**Click Here**



**Register Kit**




## STEP 2

# Prepare To Test

Hydrate! Drink plenty of water before testing. Wash hands thoroughly. Identify collection site on upper arm. Be sure area is clear of hair, shave area if necessary.

Vigorously rub testing site on arm with hand OR rub arm with included heat pack for 2 minutes to improve circulation and blood flow.

Clean testing site on arm with included alcohol wipe and let dry.

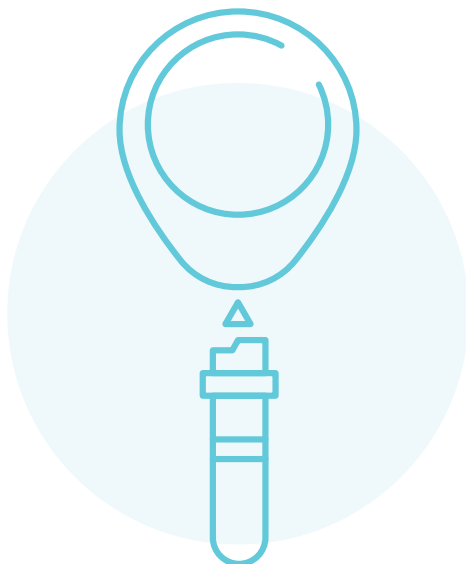

### STEP 3

## Prepare Tasso+ Quick Draw Device

Remove the Tasso+ from its packaging and unscrew the test tube top from the collection tube included.

With fill lines facing out, press tube into Tasso+ device until snug.

Remove plastic lid over the red button and peel off white paper on back of Tasso+.

Next, stick device to your upper arm. Hang arm straight down at side.



#### STEP 4

## Begin Blood Sample Collection

Firmly press red button all the way down and release.

1-2 mins may be needed for blood to appear.

Collection tube should fill up within 5 minutes.

Remove device if blood reaches top of tube before time is up. Keep arm straight down during process.



## STEP 5

# Remove Device & Collection Tube


Once you've filled the tube, remove device by peeling from one side.

Stop bleeding with supplied bandage.

Remove tube with a slight twist and pull downward.

Firmly re-apply cap onto the tube.

Be sure cap is on tight. Invert/shake the tube up and down 10 times.

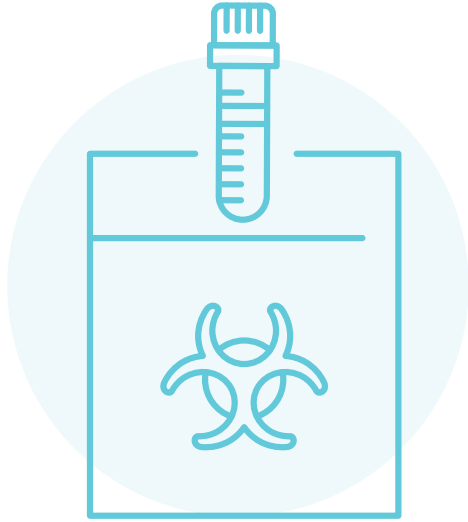


## STEP 6

# Place Blood Sample Collection Tube in Biohazard Bag

You did it! Now, place the tightly capped sample tube into the biohazard bag and seal.



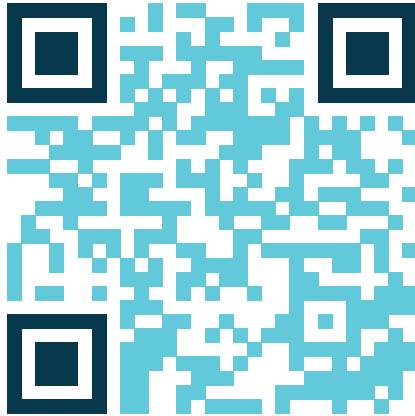

**Return Kit**

## STEP 7 Return Your Kit

Scan the QR Code above or [Click Here](#) to see all of  
your return options!



Remember to return your kit the same day you  
collect your sample.

Thank you for choosing MyLabsDirect  
for your Lab testing needs!

# You're Done!

**Questions? We've got answers.**

Call or text 877.355.7978

Email [info@mylabsdirect.com](mailto:info@mylabsdirect.com)

